



www.pacificears.com | info@pacificears.com

AU: info@pacificears.com.au
1300 90 50 10

NZ: info@pacificears.co.nz
07 542 3659



For a fitting or cleaning demonstration
please visit our YouTube channel.



Fitting Your Custom Swim Plugs

The logos on your Swimplugs are coloured, red and blue. The plug with the red logo fits the right ear, the plug with the blue logo fits the left ear. Red swimplugs have a blue logo for left and a white logo for the right. Blue swimplugs have a white logo for left and a red logo for the right. Blue and Red swimplugs: Blue is for the left side and Red is the right.

Hold the plug between your thumb and forefinger as shown (Fig1) and offer it up to your ear. It is important to make sure that the piece at the top of the mould is tucked under the flap of skin at the top of your ear. At the same time place the tip of the mould into your ear canal (Fig2). Gently twist and push, easing the earpiece into the ear canal until it locks comfortably into the ear (Fig3). When fitted, the Swimplug will fill the bowl of the ear and feel comfortable (Fig4). The supplied comfort cream can help with insertion by spreading a small amount onto the tip of the earpiece. Ensure that the Swimplug is completely flush in your ear so that a water tight seal is achieved.

Cleaning Your Swim Plugs

Your Swimplugs may be washed in lukewarm water to clean them. Please ensure that you allow them to dry fully before use.

When not in use, always store the earpieces in the pouch provided.

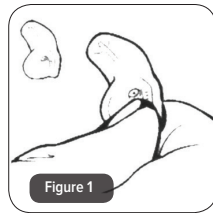


Figure 1

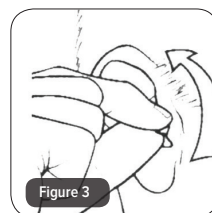
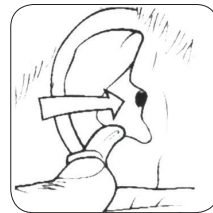


Figure 3

We want you to be delighted with your Pacific Ears Swim Plugs. If, for any reason, you experience difficulties or are not satisfied with your product or the service you received from us, please let us know by calling our customer services team on 07 542 3659 (NZ) or 1300 90 50 10 (AU) or email us at info@pacificears.com

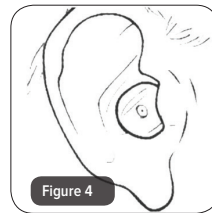


Figure 4