

CARE AND MAINTENANCE OF YOUR SLEEPSOUND EARPLUGS

Your ear plugs may be cleaned in warm soapy water or by using a sterile wipe or cloth. Please ensure that your ear plugs are completely dry before use.



For a fitting or cleaning demonstration please visit our YouTube channel.



INSTRUCTION
MANUAL

INSTRUCTION
MANUAL

SleepSound
Earplugs

SleepSound
Earplugs



FITTING GUIDE

Each earpiece is marked with a red or blue or white logo. The earpiece with the RED (logo) fits the RIGHT ear and the one with BLUE (logo) fits the LEFT ear.

Hold the earpiece as in Fig 1 and insert into the ear canal, twisting it backwards while applying slight pressure as in Fig 2 & 3. Press the earpiece until it fits comfortably in your ear as in Fig 4.

To remove, grip the earpieces and twist in a forward motion while pulling gently from the ear.

A tube of comfort cream is provided which can help with insertion, by spreading a small amount on the tip of the earpiece that fits into the ear canal.

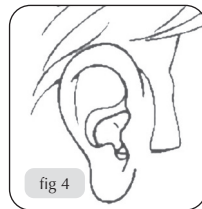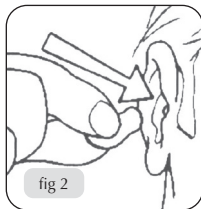

We want you to be delighted with your Pacific Ears hearing protectors. If, for any reason, you experience difficulties or are not satisfied with your product or the service you received from us, please let us know by calling our customer services team on 07 542 3659 (NZ) or 1300 90 50 10 (AU) or email us at info@pacificears.com



www.pacificears.com | info@pacificears.com

AU: info@pacificears.com.au
1300 90 50 10

NZ: info@pacificears.co.nz
07 542 3659

